

Direct Bonding



Etch enamel and dentin for 15 seconds. Rinse for 5 seconds and gently air dry. Do not desiccate. The use of SensiGuard™ following etching is optional. See IFU for details.



Apply a puddle coat of Rodin™ Bond with an applicator tip and gently agitate for 10 seconds.



Thin/dry for 10 seconds using 1/4 to 1/2 air pressure. Light cure for 10 seconds.



Place composite and light cure.



Indirect Bonding



1 Etch enamel and dentin for 15 seconds. Rinse for 5 seconds and gently air dry. Do not desiccate.



2 Apply SensiGuard™ gently for 15 seconds. Air dry for 5 seconds.



3 Apply a puddle coat of Rodin™ Bond with an applicator tip and gently agitate for 10 seconds. Thin/dry for 10 seconds using 1/4 to 1/2 air pressure. Light cure for 10 seconds.



4 Select appropriate shade of resin cement. Refer to manufacturer's instructions.



5 Gently seat restoration. Remove excess cement.



6 Light cure all surfaces.

